F-22 rollator
F-22 rollator
ambulateur F-22

item #s:

GV974BK-T

GV974RD-T

PATRIOT
★★★★★ **SERIES**
————— *drive* —————

www.drivemedical.com

parts list

Item # GV974RD-T

WEIGHT CAPACITY: 300 LBS (136 KG)

- | | |
|-----------------------------------|------------------------|
| 1. Handgrip | 7. Seat |
| 2. Adjustable tubing handgrip | 8. Bag |
| 3. Rear reflector on handgrip | 9. Front wheel |
| 4. Brake lever | 10. Brake adjust screw |
| 5. Interlock button for handgrips | 11. Rear wheel |
| 6. Back Rest | 12. Safety bow |

introduction


The Walking aid is suitable for a single user with limited mobility. The F-22 Rollator is ideal for indoor and outdoor use.

F-22 Rollator includes:

1. Bag
2. Back Rest
3. Operating Instructions

unfolding the F-22 Rollator

Position the F-22 Rollator on its wheels. Open the red safety bow and lift up (figure 1). Press both pipes from the seat downwards until they interlock (figure 2).

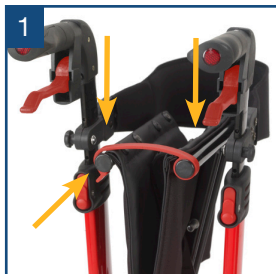height adjustment

Press the red release buttons for handgrips below, as shown in figure 3, and pull the handgrip upwards at the same time. As the handle moves, release the button. This automatically locks in the next possible height. Repeat this process until the desired height is reached.

removing / attaching the carry bag

To remove bag, twist knobs located on the front end of the seat frame outward (Figure 4 & 5). Pull out knobs and slide bag out of receptacle.

To replace bag, fold the F-22 Rollator slightly and reverse above steps.

braking / locking

Pull both brake levers upwards to apply the brakes while the F-22 Rollator is moving (see figure 6). Always brake slowly and steady.

Press both brake levers down to apply the locks while the F-22 Rollator is stationary (see figure 7). Pull up the brake levers and release to release the locks.

The effect of the brakes can be affected by abrasion of the wheels.

Regularly test the brakes.

adjustment of brakes

To adjust the braking strength, adjust the brake by pressing the screw housing downwards (figure 8) and then turn left or right (figure 9).

sitting

Position the F-22 Rollator on an even and solid surface. Lock the brakes (parking position). The F-22 Rollator must stand and must not slip. Sit straight up and do not lean backwards. (See figure 10).

DO NOT MOVE ROLLATOR WHILE SEATED.

folding / transport / storage

To fold, pull seat upwards by the handle on the seat plate as shown in figure 11. The F-22 Rollator can now be folded complete and secured by using the red safety clasp as shown in figures 12 and 13.

safety instructions

- Use only as a walking aid!
- Use only on even and solid surfaces.
- Maximum user weight: 300 lbs.
- Regularly check the screws and fasteners on the F-22 Rollator and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 11 lbs.
- Do not expose the product to temperatures above 104°F for an extended period of time, as this can cause the F-22 Rollator parts to overheat and burn the user.
- Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace tires when significantly worn or damaged.
- **DO NOT MOVE ROLLATOR WHILE SEATED.** This is a walking aid only and is not to be used as a transportation device.

cleaning / maintenance

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

lifetime limited warranty

The frame of your Drive Medical product is warranted to be free of defects in material and workmanship for the lifetime of the original consumer purchaser. The brake handle and housing are covered for a period of 5 years of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

Warning: The items and packaging in this shipment contain chemicals known to the state of California to cause cancer, birth defects or other reproductive harm.

*This product meets or exceeds all federal safety and environmental regulations.

© 2014 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in Taiwan

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© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.